


Blue Suture Grasper

For Closing Trocar incisions – Lifting up Hernia Mesh –
Ligating Abdominal Wall bleeders – For Percutaneous Suturing

- 14 Gauge Needle x 15cm long
- Grasps from 2 sides
- Easy to catch suture
- Reduces procedure time
- Reduces risk of port site Hernia's when closing port sites with below technique

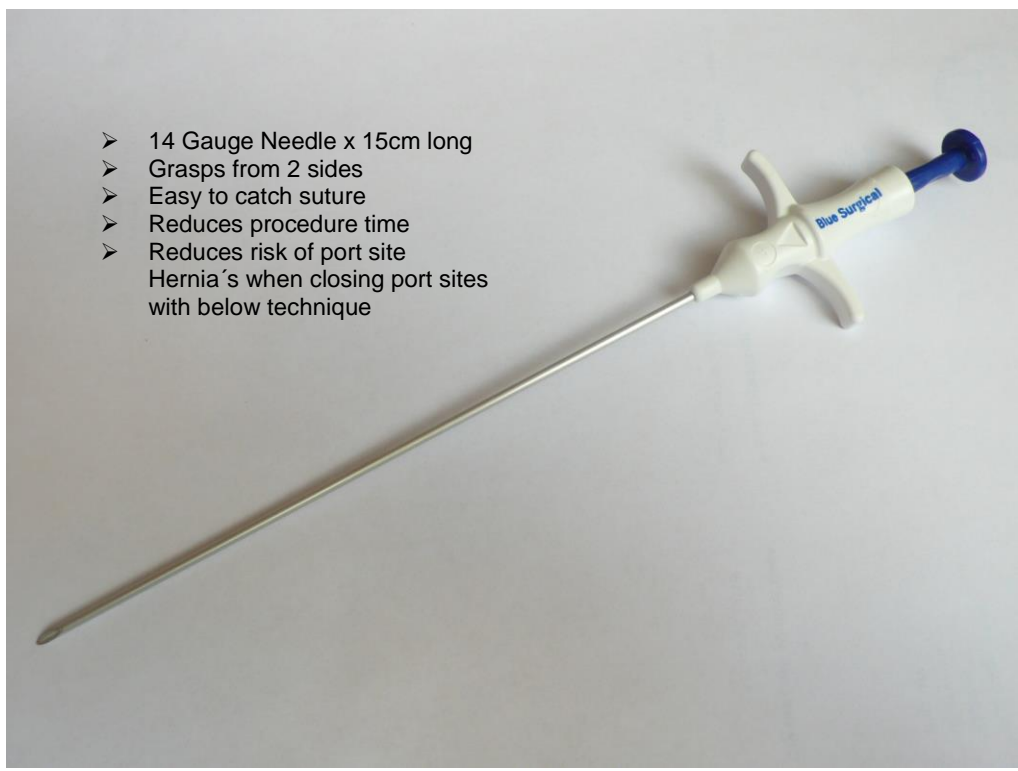


Jaws are advanced to grasp suture. Releasing the plunger retracts the jaws and suture into the needle shaft.



With the Trocar positioned to allow needle placement, the needle shaft is advanced through the peritoneum. Under direct vision, the jaws are advanced, releasing the suture loop inside the abdominal cavity.



On the opposite side of the Trocar site, the needle shaft is advanced under direct vision into the abdominal cavity. The jaws are opened to capture the suture loop.

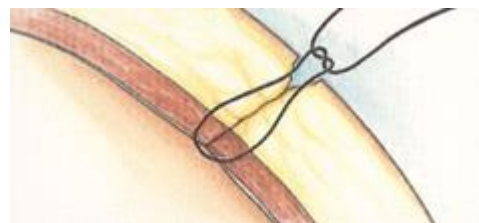

The suture loop is withdrawn from the tissue, and standard technique is used to tie the suture for Trocar site closure.

Ordering info:

REF: RSG-14-13 Blue Suture Grasper, Single use – 10 pcs. pr box

Exclusive Distribution in Europe by:

Blue Surgical ApS
International Sales Division
Virumvej 64 – 2830 Virum – Denmark
tel: + 45 21 19 22 30 – fax: + 45 45 85 22 39
e-mail: info@bluesurgical.com

CE 0086